NUTRICIA

addresses

DK

• **Nutricia A/S**

Rørmosevej 2A, DK-3450 Allerød

FI

• **Nutricia Oy**

Linnankatu 26 A, 20100 Turku
puh. (02) 274 4111 • faksi (02) 274 4100
info@nutricia.fi • www.nutricia.fi

Gr

• **Nutricia**

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flocare[®] mM 200i

INTRODUCTION

- The Flocare® MicroMAX 200i pump (mM 200i pump) is a small, lightweight pump for both portable and bedside use. The pump is based on the rotary peristaltic principle and is intended for enteral use only.

These instructions for use will guide you in a very quick and practical way through all the features of this pump.

- A more detailed technical description of the pump can be found in the Pump Service Manual which can be obtained from our local sales organisation/Nutricia subsidiary (see back of the booklet).

PREPARATION

- Check the integrity of the pump. Do not use the pump if it is damaged. If a technical failure occurs or if the pump is dropped, the pump should be checked by a qualified technician.
- Do not use the pump in areas where there is a risk of explosions e.g. in the presence of flammable anaesthetics.
- If the pump has been stored for any period of time, it should be plugged into the mains to recharge the battery before commencing enteral feeding. The battery will be completely charged after 4 hours.
- The mM 200i pump should only be used in combination with the appropriate Flocare® MicroMAX pump set! (see enclosed list at the end of this booklet).
- Check the position of the feeding tube, as advised by your healthcare professional, before commencing tube feeding.
- Pump fed patients should be regularly monitored and supervised.
- For bedside use, the pole clamp can be attached to the pump with the screw provided.

SERVICE AND WARRANTY

Only authorised personnel should perform all service work for pumps under warranty. A technical manual is available on request, which includes servicing instructions, electrical layout, assembly charts and a component chart. For service and repair of pumps under warranty please contact the authorised service centre of your local Sales Organisation/Nutricia subsidiary (see address at the back of the booklet).

Limitations of warranty

Nutricia warrants the Flocare® MicroMAX 200i Enteral Feeding Pump against all defects in material and workmanship under normal use for a one year period, from date of delivery, to the purchasing authority or a person acting on the authority's behalf. This warranty is only valid to the original purchaser and does not extend to any pump or part thereof, which has been subjected to an accident, spillage or mistreatment.


In addition, the warranty will not apply

- if the Flocare® mM 200i pump has been repaired, adjusted or adapted by anyone other than personnel authorised by Nutricia.
- if the pump has not been operated and maintained according to the instructions.
- if the serial number has been erased or changed.

Pumps requiring service or repair, should be returned to our local service organisation (see address at the back of the booklet) properly packaged, postage pre-paid with details of fault. Loss or damage in return shall be at the purchaser's risk. Nutricia reserves the right to repair or replace (at its sole option) any Flocare® mM 200i Pump, which fails to meet the foregoing warranty.

OPERATING INSTRUCTIONS

Insertion of the Flocare® mM pump set

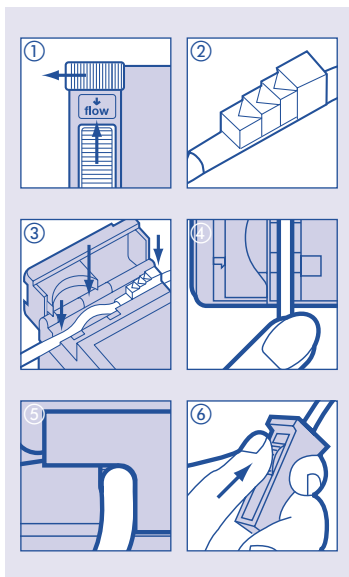- Connect the feed container as instructed on the packaging of the Flocare® MicroMAX pump set. Fill the pump set completely with feed and close the roller clamp.
- Open the pump-door by first moving the top safety lock to the left and at the same time sliding the bottom lock up ①.
- Take the pump set and insert the retainer section with the three arrows ② between the sensors of the pump ③. Note the arrows indicate the direction of feed flow through the pump and giving set.
- Guide the silicone tubing over the rotor and insert into the notch underneath ④.
- Close the pump door, make sure the tube is resting in the notch on the underside of the pump ⑤. Fully open the roller clamp ⑥.

Switching "ON"

Press the "on/off" key.

The pump carries out a self-test that ends with a short beep and switches into the hold or pause position.

Switching "OFF"

Press the "on/off" key and hold it down for 3 seconds. An intermittent alarm will be heard and the pump switches "off".

The feeding programme (installed parameters) will be retained in the pumps memory for 24 hours.

"HOLD" mode

To switch the pump into "hold" mode whilst operating, press the "on/off" key once. Three arrows appear in the display and the pump stops working.

The programmed flow rate and the volume fed are retained.

The "HOLD" mode is used:

- to change the flow rate without switching pump off,


- to change the feed container,
- to administer medication.

After 3 - 6 minutes an audible alarm (6 beeps) sounds and the three arrows flash and then every 15 minutes to remind the user that the program is paused.

To restart pump from the pause or hold position, press "on/off" again.

Display

The pump has a liquid crystal display (L.C.D.) with 2x16 character positions and back light. The following information can be found in the display: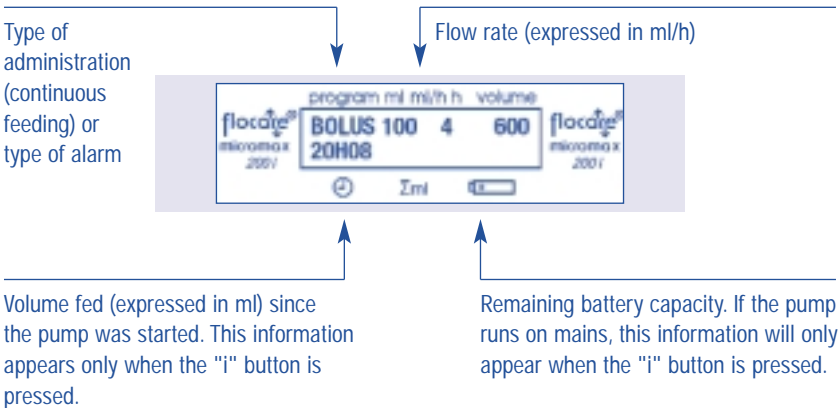


Programming the pump

The mM 200i pump can be used for continuous, intermittent or bolus feeding.

Switch the pump "on". After the self-test, the last installed feeding program appears on the display and three black digits start blinking in the display.

Continuous feeding

Press the "continu" key followed by the "+" or "-" key to set the flow rate.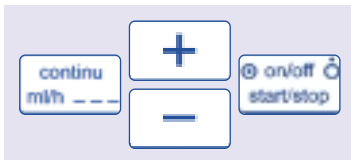

The range goes from 5-400ml/h in 1ml increments.

Start the pump by pressing "on".

The arrow in the display starts moving.

The flow rate is displayed whilst functioning.


Intermittent feeding

Press the "continu" key. The flow rate will be replaced by 3 blinking black digits. Press the "+" or "-" key to set the flow rate.

Press the "volume" key and install the volume to be administered with the "+" and "-" key.

The range goes from 1 - 4000ml.

The feeding time in hours and minutes will automatically appear in the display. Start the pump by pressing "on".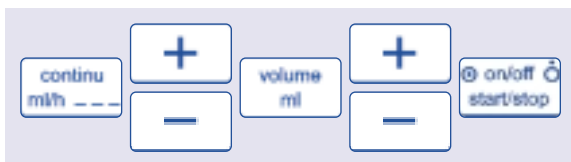


Bolus administration

Press the "bolus" key. The word "bolus" appears on the display and 3 black digits start blinking. Press the "+" or "-" key to set the bolus volume between 1 - 500ml.

Select the interval time (time of feeding + rest period) by pressing the "interval" key and thereafter the "+" or "-" key to install the interval time between 1 and 9 hours. The bolus speed is fixed at 800ml/h.

The pump can now be started, or if required the total volume to be administered can be programmed by pressing the "volume" key and then the "+" or "-" key until the required total volume has been reached. (total volume 1 - 4000ml).

The total feeding time in hours and minutes will then appear automatically on the display. Start the pump by pressing "on".

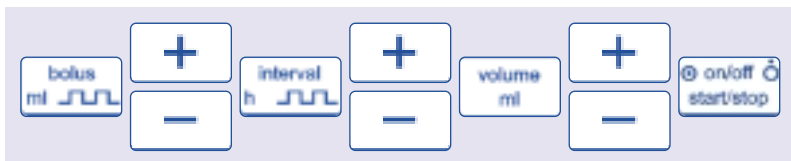Keep tube open feature

The keep tube open feature (K.T.O.) starts a flow rate of 5ml/h at the end of a programmed volume, when the Flocare® Micromax has automatically stopped. In practice this works only in combination with an intermittent or bolus program.

Information key

Press the "i" key during feeding and the LCD display will give the following information:

- Volume of feed given, since the start of the programme.
- Remaining time required for completing the programme.

If the pump is functioning on batteries, the capacity is permanently displayed.

The display automatically returns to its first screen after 30 seconds or by pressing the "i" key a second time.

To change the feeding programme during functioning

Press the "start/stop" key.

Change the programme by pressing the required key and adjusting with the "+" or "-" keys.

Restart the pump by pressing the "on" key again.

Battery operation

- The mM 200i pump is equipped with a NiCd battery.
- The battery symbol automatically disappears from the display when the adapter is plugged in. The battery will be completely recharged within 4 hours, whether the pump is switched "on" or "off".
- Once recharged the pump can operate on battery for approx. 20 hours or 4000 ml of feeding.
- In case of power failure the pump automatically switches to battery powered operation.
- When the battery capacity is at 10%, an alarm beep can be heard and is repeated every 10 seconds for approximately 1-minute. Simultaneously, the battery symbol on the display will blink.

After this alarm, the pump will function on batteries for approximately 2 hours (depending on the flow rate).

When the battery capacity is at 5%, the alarm will start again working every 10 seconds and the battery symbol will blink. The pump will keep working normally. Connecting the pump to the mains or switching the pump off will stop the alarm.

When the battery capacity is at 0%, the alarm will start again beeping every 10 seconds and the battery symbol will blink.

The pump will stop working after a further 15 minutes. The pump must be immediately connected to the mains.

- The lifetime of the battery is optimally maintained by using the pump on a regular basis on battery until it is fully discharged and recharged.

Connection to an external alarm system

- The serial connector at the side of the pump offers the option of connecting the pump via the Flocafe® Nurse Call (corporate code 35722) to an external alarm system.
This external alarm system may be with open or closed contact.
- Follow the instruction for use of the Flocafe® Nurse Call to connect the pump.

ALARM FUNCTIONS AND SAFETY FEATURES

In case of any of the problems listed on the problem solver chart, the pump delivers an audible and visual alarm and stops working. The back light of the LCD automatically switches on. Exceptions to this are the "low battery alarm", (<10%, <5%, 0%) and "maintenance request" in either case the pump continues working.

Action in the event of an alarm:

- Check the type of alarm indicated on the LCD.
- Press the "on/off" key to stop the audible and visual alarm.
- Correct the cause of the alarm as described in the table.
- Start the pump again by pressing "on/off"

PROBLEM SOLVER CHART

Condition	Cause	Correction
 <i>Moving arrows</i>	The pump works.	<ul style="list-style-type: none"> • Normal pump function.
 <i>Blinking arrows</i>	Pump remains in the "hold" mode for more than 3 minutes.	<ul style="list-style-type: none"> • Start the pump by pressing "on/off". • Or put the pump in "hold" mode again.
 <i>Blinking number</i>	Volume programmed has been administered.	<ul style="list-style-type: none"> • Press the "stop" key.
 DOOR	The door is unlocked.	<ul style="list-style-type: none"> • Stop the alarm by pressing "on/off". • Open the door and close again. • Restart the pump.
 OCC	The flow is interrupted.	<ul style="list-style-type: none"> • Stop the alarm by pressing "on/off". • Open the roller clamp. • Check there are no kinks in the tubing. • Check the feeding tube and set are clear of blockages. • Restart the pump.
 AIR	The feeding container is empty.	<ul style="list-style-type: none"> • Stop the alarm by pressing "on/off". • Replace with a full container of feed and flush the set. • Restart the pump.
	Air bubbles in the feeding set.	<ul style="list-style-type: none"> • Stop the alarm by pressing "on/off". • Disconnect the set from the tube and flush the line. • Restart the pump.
	The set not fitted or wrongly fitted in the pump.	<ul style="list-style-type: none"> • Stop the alarm by pressing "on/off". • Open the door and check that the retainer segment is in the correct place between the air detection sensors of the pump. • Restart the pump.
 10%	The battery capacity is too low.	<ul style="list-style-type: none"> • Charge the pump for 4 hours through the mains.
> 10 % deviation on the flowrate	Feeding set not changed often enough.	<ul style="list-style-type: none"> • Change the feeding set every 24 hours.
LED on the adaptor does not light up.	The adaptor is not connected to mains.	<ul style="list-style-type: none"> • Connect the adaptor to mains supply.
	The adaptor is defective.	<ul style="list-style-type: none"> • Adaptor must be replaced.
 ERROR	Internal electronic malfunction.	<ul style="list-style-type: none"> • The pump must be inspected by technical service personnel. • Adaptor must be replaced.
"Maintenance Request"	The pump administered > 600 litres.	<ul style="list-style-type: none"> • The pump must be inspected by technical service personnel.

MAINTENANCE

Cleaning:

- Always unplug the pump prior to cleaning to avoid electric shock hazard.
- On a regular basis thoroughly clean all surfaces with a warm damp cloth and finally with a water based disinfectant.
- If feed is spoiled on the air sensor or rotor, immediately, carefully clean using a cotton bud. Do not run under tap water!
- Always maintain the four rollers on the rotor in a clean state to ensure they spin freely. Prevent seepage of water into the pump.
- Do not immerse the pump in water or disinfectant.

Safety notes:

- There are no user parts inside the pump. Only authorised personnel may open the pump.
- Respect the annual service interval.
- If any fault occurs during use, or if the pump is dropped, it should be checked by authorised technical personnel prior to use.

TECHNICAL SPECIFICATIONS

- This MicroMAX pump in combination with the adapter (FW 7202 or FW 7555M) and the Flocare® Nurse Call are designed to EN60601-1-2 and are in compliance with Directive 93/42/EEC.

CE 0344

- IEC 601: Class II 

Medical Device Directive: Class IIa

- Microprocessor controlled

Charger: input  100-240V AC 50-60Hz - 400 mA
output 12 V = / 1,25A

- Battery: internal rechargeable NiCd battery, 6V---800 mAh

- BF Equipment 

- Dripwaterproof IPX 1

- Case material: PC-ABS plastic, flame retardant

- Dimensions: 146 x 75 x 43 mm

Weight: approx. 500 g

- Accuracy flow rate: $\pm 6,5\%$ with appropriate Flocare® mM pump set

- Humidity: Operation mode: 30 - 75%

Storage – non condensing: 10 - 90%

- Temperature: Operation mode: 0 - +40°C

Storage and transportation: -40 - +50°C

- Atmospheric pressure: Operation mode: 70 - 106 kPa

Storage and transportation: 50 - 106 kPa

- Occlusion detection pressure: 150 kPa ($\pm 20\%$)

- Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents.

- The use of other accessories, transducers and cables than listed above may result in increased emissions or decreased immunity of the equipment of the Flocare® MicroMAX pump.

- Portable and mobile RF communications equipment (cellular telephones) can affect medical electrical equipment.

If the Flocare® mM200i pump is used adjacent to or stacked with other equipment, the pump should be observed to verify normal operation.

flocare[®] mM 200i